
DKL GmbH

An der Ziegelei 4 · D-37124 Rosdorf
Fon +49 (0)551-50 06 0
Fax +49 (0)551-50 06 299
www.neuepolster.de · info@dkl.de

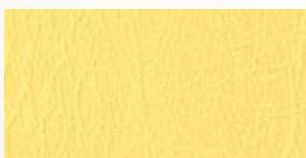saharagelb, saharayellow, jaunesahara 326


kakao, cocoa, cacao 109

lindgrün, lime green, lvert tilleul 329

stahlblau, steel blue, bleu acier 221

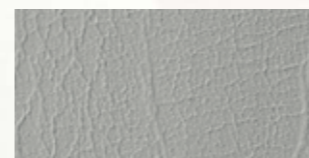grau, grey, gris 125

mango, mango, mangue 327


flieder, lilac, lila 220

mintgrün, mint, vert menthe 223


ozeanblau, ocean blue, bleu océan 328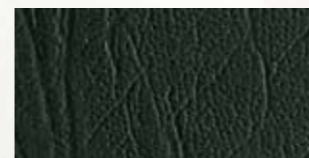

anthrazit, anthracite, anthracite 112

sand, sand, sable 103

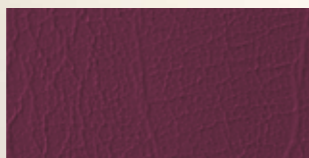atlasrot, plum, rouge atlas 214

grün, green, vert 127

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PLEASE NOTE: We do not accept any liability for discolourations caused by textiles on white, light-coloured, yellow, light green, light blue, beige and orange upholstery parts.

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ProLINE

Properties:

This material has a leather-like surface with a skin-friendly coating. It is waterproof and can be disinfected.

Care Instructions:

Never use disinfectants containing alcohol. We recommend applying our spray foam proPad to your upholstery. Clean the upholstery in circular motions with a soft sponge and lukewarm water at the end of each treatment day. Never use cleaning agents or household cleaners. After being cleaned, the upholstery should be wiped dry with a cloth. Then apply DentaProtect (wipe dispenser), our sealant for imitation leather.

Soiling / Discolouration

Light colours such as white, yellow, light green, light blue, beige or orange, are particularly susceptible to soiling or discolouration. Residual humidity from disinfectants enhances the migration of colourants from textiles. Particularly aggressive discolouration stains are caused by belts and suede leather clothes. Carefully wipe the upholstery dry after the residence time of the surface disinfectant! Soilings can be removed with water and a sponge or a brush. Please note: discolourations caused by jeans or other textiles excluded from manufacturer's guarantee.

You will find detailed information on this topic in the DKL care brochure, which is available for download on our website www.neuepolster.de

Please note that, for technical reasons, the colours may differ slightly from the original surfaces.

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